# BébéSounds®

## Angelcare® Deluxe Movement Sensor with Sound Monitor

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[www.bebesounds.com](http://www.bebesounds.com)

# TABLE OF CONTENTS

Introduction.....	3
Setting Up Your Monitor.....	4
Changing Optional Features Settings.....	8
Testing Your Monitor.....	11
Using Your Monitor.....	12
Special Features.....	12
Tips for Using Your Monitor.....	14
Understanding the Symbols.....	14
Summary of Alarms.....	16
Important Safety Instructions.....	16
Care and Cleaning.....	17
Troubleshooting.....	17
Technical Specifications.....	20
Limited Product Warranty.....	21
Also Available from BébéSounds®.....	22
En Español.....	23

## INTRODUCTION

Thank you for purchasing our Bébésounds® Deluxe Angelcare® Movement Sensor with Sound Monitor. This monitor can detect your baby's slightest movements, even while she sleeps. An alarm will sound alerting you to check your baby ONLY if she becomes absolutely still for 20 seconds. In addition, your Angelcare® monitor can also work as a conventional sound monitor.

*Please read all the instructions carefully before operating this monitor. Please save this manual for future reference.*

If you have any questions, please call the Bébésounds® Help Desk toll free at 1-888-232-6476 Mon-Fri from 9:00AM-5:00PM EST.

**WARNING:** This monitor is not intended to replace adult supervision of children. All children should be monitored at all times by responsible adults.

**WARNING:** This product is not a toy. Do not allow children to play with it. Keep out of reach of children.

### Contents

- 1 Movement Sensor Pad
- 1 Nursery Unit (Transmitter)
- 2 Parents' Units (Receiver)
- 2 Charging Bases
- 3 AC Adapters
- 8 AAA Rechargeable batteries

### This monitor offers the following features:

- Movement Sensor (Pg. 8)
- Sound Monitor (Pg. 8)
- "Tic" feature (Pg. 8)
- Out-of-Range Indicator (Pg. 9)
- Voice Activation with Sensitivity Control (Pg. 9)
- Sound Alarm (Pg. 10)
- Vibration Alarm (Pg. 10)
- Room Temperature and Temperature Control (Pg. 10)
- Sound Lights (Pg. 13)
- Movement Indicators (Pg. 13)
- Temporary Mute (Pg. 13)
- Parents' Unit Locator (Pg. 13)
- Nursery Nightlight (Pg. 13)

## SETTING UP YOUR MONITOR

### SENSOR PAD (See figs. 1 and 2)

Place the Sensor Pad under the center of the crib mattress with the turquoise side facing up. Do not place any loose bedding or linen between the crib mattress and the Sensor Pad. The Sensor Pad must rest on a flat, firm surface. If your crib has a spring or slat base, place a piece of plywood or Masonite under the Sensor Pad. This support must measure at least 13 inches long on all four sides and  $\frac{1}{4}$  inch thick. (You may order a Masonite board directly from us. See Pg. 22). If your crib has a plywood base, your mattress support is firm enough to use the monitor properly.

*Only use your Angelcare® monitor in a stationary sleeping environment such as a crib where the Sensor Pad can rest on a flat, firm surface. Be sure to place the turquoise side of the Sensor Pad facing up.*

**CAUTION: To avoid potential strangulation accidents resulting from loose cords or wires in the crib, make sure the Sensor Pad cord passes underneath the mattress entirely. After positioning the Sensor Pad under the mattress, bring the cord to the corner of the crib and secure it to the top and bottom of the leg of the crib, or wind it around the leg (see fig. 1).**

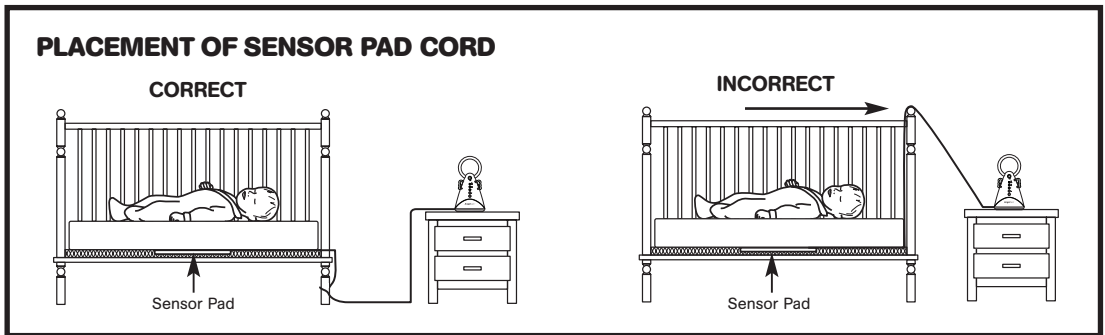
fig. 1

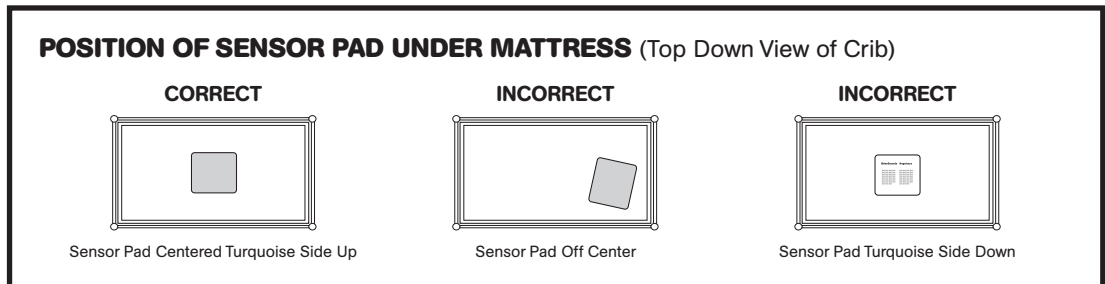
fig. 2

**NURSERY UNIT (Transmitter) (See figs. 3 and 4)**

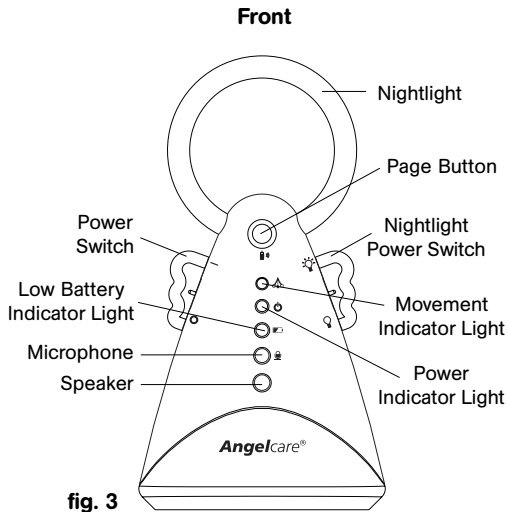
fig. 3

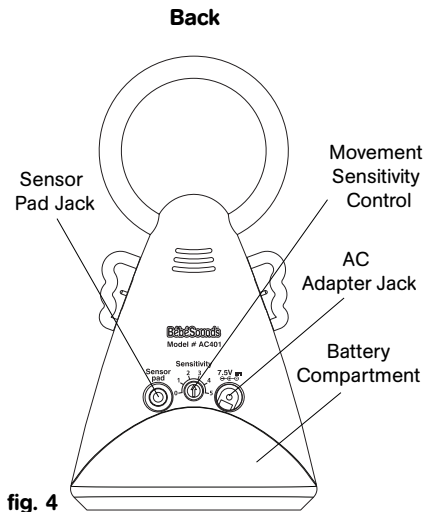
fig. 4

**AC Power.** The Nursery Unit can operate with just batteries or an AC adapter. For AC power insert the AC adapter into the jack marked “7.5V” and plug the adapter into an electrical outlet. If you choose to use an AC adapter, we suggest you also keep batteries in at all times as it will automatically convert from AC to battery power during a power outage. However, if you do not keep batteries in the unit, it will operate on just AC power.

**Battery Power.** For battery power the Nursery Unit requires four AAA alkaline or rechargeable batteries (not included). Rechargeable batteries can be used but they will NOT be recharged in the Nursery Unit by plugging in the AC adapter. They will have to be removed and recharged in an external charger.

To install or change batteries, remove the battery cover on the bottom of the Nursery Unit. Insert four AAA batteries matching positive (+) to positive (+) and negative (-) to negative (-) and replace the battery cover. Replace the batteries when the red Low Battery Indicator flashes on the Nursery Unit. The Parents’ Unit will also alert you when the batteries need to be replaced in the Nursery Unit. The Battery symbol will flash, the Nursery Unit symbol will appear (see fig. 5), and a beep will sound every 15 seconds for approximately 30 minutes until the Nursery Unit shuts down. If the Out-of-Range feature is on (refer to Pg. 9), the Out-of-Range symbol will flash on the Parents’ Unit once the Nursery Unit shuts down.

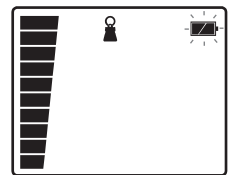
fig. 5

For long periods of non-use, we suggest removing the batteries and unplugging the AC adapters.

**IMPORTANT:** Only use the AC adapters included with your monitor. Do not use these adapters with any other equipment.

**Movement Sensitivity Control.** The movement sensitivity control is on the back of the Nursery Unit. The level has been preset to 2.5 (see fig. 6). Do not adjust this unless you receive false alarms. Refer to the Movement Sensitivity Adjustment section on Pg. 12. **It is extremely important to follow our instructions for setting the sensitivity level.**

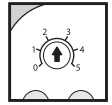
fig. 6

Now plug the Sensor Pad cord into the jack on the back of the Nursery Unit marked "Sensor Pad." Turn the Nursery Unit on using the switch on the left. The green Power Indicator will light up.

**PARENTS' UNIT (Receiver) (See figs. 7, 8, 9, and 10)**

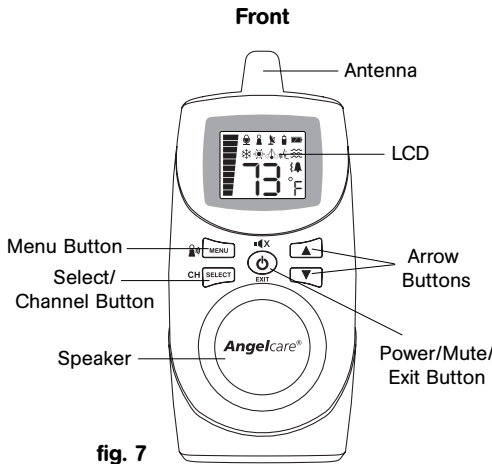
fig. 7

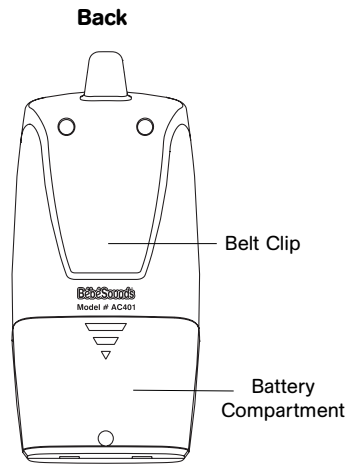
fig. 8

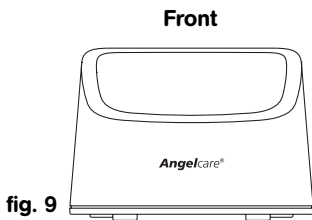
fig. 9

**Charging Base**

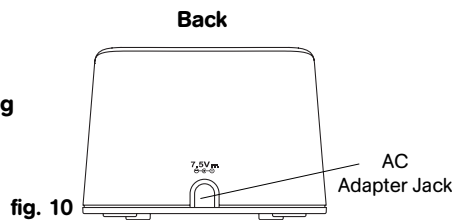
fig. 10

**IMPORTANT:** You must use only rechargeable batteries in the Parents' Unit. **DO NOT** use alkaline batteries or you will severely damage the unit. Charge the batteries for 15 hours prior to first use. Be sure to have the monitor in the OFF position during this period. Only use the AC adapters included with your monitor. Do not use these adapters with any other equipment.

The Parents' Unit requires four AAA rechargeable batteries (included). To insert batteries, remove the battery cover on the back of the unit and insert the batteries matching positive (+) to positive (+) and negative (-) to negative (-) and replace the battery cover. Insert the AC adapter into the jack marked "7.5V" on the back of the Charging Base and plug the adapter into an electrical outlet. Place the Parents' Unit in the Charging Base.

When charging is complete, turn the Parents' Unit on by pressing the Power button for two seconds. The display will turn on and the channel will be displayed for 10 seconds. Then the temperature of the nursery will be displayed. (If this fails to happen, you may need to reestablish the connection between the Parents' and Nursery Units. Please refer to the Parents' Unit and Nursery Unit Connection section on Pg. 13.) If the Parent's Unit is turned on while the Nursery Unit is off, the Nursery Unit symbol will flash and "--" will appear. After one minute, the Parents' Unit will beep and shut down automatically. To manually turn off the Parents' Unit, press the Power button for three seconds.

Recharge the Parents' Unit when the Low Battery symbol appears (see fig. 11) and a beep is heard every 15 seconds for approximately 30 minutes. After approximately 30 minutes, the unit will shut down.

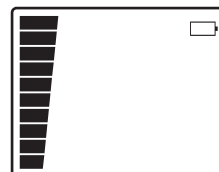
fig. 11

### Optional Feature Symbols

The monitor's optional features are controlled by the Parents' Unit and their symbols can be seen on the LCD. These features are preset to the following:

Feature	Symbol	Default Setting
Sound Monitor		ON
Movement Sensor		ON
"Tic" feature		OFF
Out-of-Range Indicator		OFF
Voice Activation		OFF
Voice sensitivity level		2
Sound Alarm		ON
Vibration Alarm		ON
Room Temperature Control		OFF
Low Temperature setting		59°F
High Temperature setting		86°F
Speaker volume level		3

If you would like to keep these settings and not make any changes, please turn to the Testing Your Monitor section on Pg. 11. If you would like to change any settings, please continue with the following section. Be sure to save this manual for future reference.

## CHANGING OPTIONAL FEATURE SETTINGS

The controls for all the optional features are located on the Parents' Units. You only need to change the settings on one Parents' Unit as the second one will automatically change as well.

To access the optional features, press the Menu button. You can toggle through the features by repeatedly pressing the Menu button. A flashing symbol indicates that feature can be activated, deactivated, or adjusted by pressing the Arrow buttons and then the Select button. The Parents' Unit symbol will also appear when you are changing settings for the Parents' Unit. The Nursery Unit symbol will also appear when you are changing settings for the Nursery Unit.

**NOTE:** If you do not press the Select button after you have made your selection, the unit will revert to the previous setting.

### Sound, Movement, and "Tic" functions

After you press the Menu button, use the Arrow buttons to toggle between these three functions until you get your desired combination. You can select one of the following five choices:

- **Movement & Sound.** This is the way your monitor is set from the factory. The Movement Sensor is enabled and detects your baby's movements. The Sound feature is enabled so you will hear the alarm and all of your baby's sounds, and see sound lights. For this setting, the Sound and Movement symbols will appear (see fig. 12).

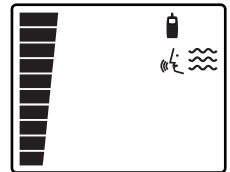
fig. 12

- **Movement, Sound, & "Tic."** The Movement Sensor is enabled and detects your baby's movements. The Sound and "Tic" features are also enabled so you will be able to hear the alarm, your baby's sounds, see sound lights, and hear a ticking sound as the Sensor Pad senses your baby's movement. For this setting, the Sound, Movement, and "Tic" symbols will all appear (see fig. 13).

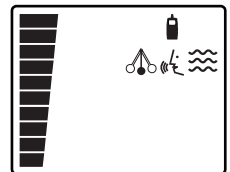
fig. 13

- **Sound.** The Movement Sensor is disabled but you will still be able to hear your baby's sounds and see sound lights. Your Angelcare® works like a regular sound monitor when set to this function. For this setting, only the Sound symbol will appear (see fig. 14).

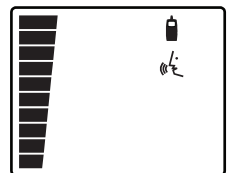
fig. 14

- **Movement.** The Movement Sensor is enabled and detects your baby's movements. If no movement is detected, the alarm will sound on the Nursery Unit after 20 seconds. Your baby's sounds, the movement alarm, and the ticking will not be transmitted to the Parents' Unit. For this setting, only the Movement symbol will appear (see fig. 15).

fig. 15

- **Movement & Tic.** The Movement Sensor is enabled and detects your baby's movements. You will also hear a ticking sound from the Nursery Unit as the Sensor Pad senses your baby's movement. Your baby's sounds, the movement alarm, and the ticking will not be transmitted to the Parents' Unit. For this setting, the Movement and "Tic" symbols will appear (see fig. 16).

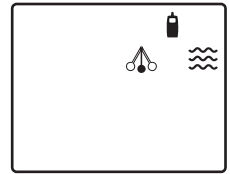
fig. 16

Once you have completed your selections, press the Menu button to change another feature setting or press the Exit button to end programming.

**Out-of-Range Indicator** (See fig. 17)

The Out-of-Range Indicator will alert you when the Parents' Unit and the Nursery Unit are not in communication with each other. This may occur when the units are too far from each other, if the Parents' Unit is on when the Nursery Unit is off, or if the units have lost communication with each other. The Out-of-Range symbol will flash on the Parents' Unit and you will hear two beeps every eight seconds.

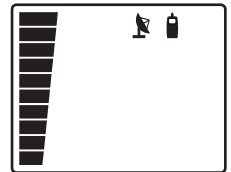
fig. 17

To turn this feature on or off:

1. Press the Menu button until the Out-of-Range symbol appears.
2. "On" or "Off" will appear below. Use the Arrow buttons to make your selection.
3. Press the Select button.
4. Press the Menu button to change another feature setting or press the Exit button to end programming.

**Voice Activation** (See fig. 18)

Voice Activation eliminates the background "hum" associated with sound monitors when there is no significant sound being transmitted from your baby's room. It does this by automatically placing both the Nursery Unit and Parents' Unit in a "standby" mode. When sound is detected, the Voice Activation automatically returns both units to an active mode and immediately transmits the alarm or your baby's sounds to you.

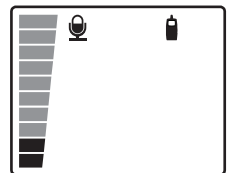
fig. 18

To use this feature:

1. Press the Menu button until the Voice Activation symbol appears.
2. "On" or "Off" will appear. Use the Arrow buttons to make your selection.
3. Press the Select button.
4. If you choose to turn Voice Activation on, press the Menu button and then use the Arrow buttons to choose the level of sensitivity. If you want to hear the slightest sound in the nursery, adjust the sensitivity to the maximum level (4). If you only want to hear when the baby cries loudly, adjust the sensitivity to lowest level (1). You may have to adjust the sensitivity level a couple of times until you get the exact level you want.  
**NOTE:** The movement alarm will be picked up at all levels of sensitivity.
5. Press the Select button when the desired level is displayed.
6. Press the Menu button to change another feature setting or press the Exit button to end programming.

### Sound and Vibration Alarms (See fig. 19)

If no movement is detected by the Sensor Pad, you have the option to have a sound alarm, a vibration alarm, or both.

1. Press the Menu button until the Sound and/or Vibration symbols appear.
2. Use the Arrow buttons to make your selection. If you want only a sound alarm, the Sound symbol will appear. If you want only a vibration alarm, the Vibration symbol will appear. If you want both alarms, both the Sound and Vibration symbols will appear.
3. Press the Select button when your desired setting appears.
4. Press the Menu button for the next option or press the Exit button to end programming.

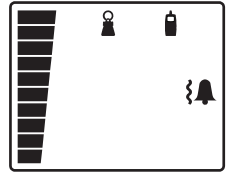
fig. 19

**NOTE:** The vibration alarm will not work when the battery level is low on the Parents' Unit. You will know when the battery level is low because the Low Battery Indicator will appear and the unit will beep. The vibration alarm also will not work when the Parents' Unit is in the Charging Base.

### Room Temperature Control (See fig. 20)

The temperature of your baby's room is always displayed on the Parents' Unit. The Room Temperature Control, however, is an optional feature. When you activate this feature you can set your desired temperature range for your baby's room. If the temperature goes above or below the range you have set, the High or Low Temperature symbol will flash and three beeps will be heard every four seconds.

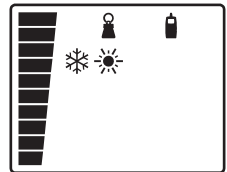
fig. 20

If the Temperature Control alarm does go off, you can temporarily turn it off for two minutes by pressing the Power button. The symbol will continue to flash but the beeping will stop. The alarm will begin again after two minutes unless the room temperature returns to your preset range or you turn the Temperature Control feature off. The preset Temperature Control range is 59°-86°F.

To set the temperature range:

1. Press the Menu button until the High Temperature and Low Temperature symbols appear.
2. "C" or "F" will appear. Use the Arrow buttons to turn it on.
3. Press the Select button.
4. Press the Menu button.
5. The Low Temperature symbol will flash. Using the Arrow buttons, select the lowest temperature you want the nursery to reach.
6. Press the Select button.
7. Press the Menu button.
8. The High Temperature symbol will flash. Using the Arrow buttons, select the highest temperature you want the nursery to reach.
9. Press the Select button.
10. Press the Menu button to change another feature setting or press the Exit button to end programming.

If you choose to turn the Room Temperature Control off, the nursery temperature will still be displayed on the Parents' Unit; however, it will not alert you if the temperature is outside your desired range.

## TESTING YOUR MONITOR

Always test your monitor before using it the first time and then on a periodic basis. Before testing your monitor, complete the setup process as described above.

- 1. Testing for Sound.** Turn the Nursery Unit and Parents' Unit on and be sure the Sound symbol appears on the Parents' Unit. Have someone else speak into the Nursery Unit or place a radio near the Nursery Unit while you take the Parents' Unit into another room. You should hear the sounds being transmitted from the Nursery Unit. If there is no sound, adjust your volume control using the Arrow buttons and/or refer to the Troubleshooting section on Pg. 18. If you experience interference or static, switch to another channel on the Parents' Unit. Press the Channel/Select button for two seconds. The Out-of-Range symbol will flash and the current channel will be displayed. Use the Arrow buttons to choose a different channel. When the desired channel is displayed, press the Select button again. The Nursery Unit will automatically switch to the same channel as the Parents' Unit. The other Parents' Unit will also automatically switch to the same channel.
- 2. Testing for Movement.** Be sure the Movement symbol appears on the Parents' Unit. Gently pat your hand on the mattress. The green Movement Indicator Light on the Nursery Unit will flash and the Movement symbol on the Parents' Unit will swing as the Sensor Pad senses movement. Remove your hand from the mattress. The Movement Indicator Light will stop flashing, the Movement symbol will stop swinging, and a pre-alarm beep will sound after 15 seconds. Five seconds later (for a total of 20 seconds), the Nursery Unit symbol will appear on the Parents' Unit display and the alarm will beep and/or vibrate continuously on both units.

To turn off the alarm on the Parents' Unit, press the Power button. This will temporarily mute the Parents' Unit for two minutes. The Nursery Unit alarm, however, will continue to sound. To turn off the alarm on the Nursery Unit, turn the unit off and then back on or pat your hand lightly on the mattress so the Sensor Pad will redetect movement. The units will then revert to their normal functions. If the Nursery Unit alarm is not turned off, the Parents' Unit alarm will reactivate after two minutes. **NOTE:** If you turn off the Nursery Unit and leave the Parents' Unit on, the Out-of-Range Indicator will flash and sound if this feature is turned on.

If the alarm does not sound when you are not touching the mattress, the Sensor Pad may be detecting a continuous vibration from another source from inside and/or outside your baby's room. Common sources of this interference are crib mobiles, fans, washing machines, or even loud music. Therefore, it is very important to make sure all vibration sources are eliminated before using the monitor. Make sure the Sensor Pad is on a flat, firm surface. Place the crib near a solid supporting wall and away from strong air currents. Do not use crib mobiles or accessories that may cause vibrations. Avoid contact with the crib when the monitor is in use.

If you find the alarm will still not sound when you are not touching the mattress or crib and you have removed all external sources of vibration, the sensitivity level may need to be decreased. Please refer to the Movement Sensitivity Adjustment section on Pg. 12.

- 3. Testing for Battery Power.** To verify that all units are operating on battery power, install batteries into the units. The Parents' Units should operate when they are removed from the Charging Bases (after the initial 15-hour charge) and the LCDs should be lit. The Nursery Unit should operate when the AC adapter is unplugged and the green Power Indicator should light up.

## USING YOUR MONITOR

After the monitor has been set up and tested, place your baby in the crib.

If you have selected the Movement and Sound feature on the Parents' Unit, the Sensor Pad will detect your baby's slightest movements and the Nursery and Parents' Unit will act as a sound monitor. If you have only selected the Sound feature, it will not detect your baby's movement but will continue to work as a regular sound monitor and only transmit nursery sounds.

If you have selected the Movement feature, be sure to turn off the Nursery Unit before you remove your baby from the crib. If you forget to do this, the alarm will sound after 20 seconds.

### Using Your Monitor for More Than One Baby

The Angelcare® monitor can only be used to monitor the movement of one baby at a time. If you have twins or you want to monitor two babies at the same time, you will need to purchase two complete units. Please remember the babies will have to sleep in separate cribs and the two sets of monitors must be on different channels to work properly. You can use a maximum of eight Angelcare® monitors for up to eight babies in one household since the monitor has eight channels.

## SPECIAL FEATURES

### Movement Sensitivity Adjustment

The movement sensitivity adjustment dial has been preset to level 2.5 which will suit most situations. The highest sensitivity level is 5 (most sensitive) and the lowest sensitivity level is 1 (least sensitive).

You may get false alarms if your baby is in a very deep sleep or has moved to the edge of the crib. If you get false alarms due to these circumstances, you should increase the sensitivity level by using the sensitivity dial on the back of the Nursery Unit.

**NOTE:** Adjust the sensitivity level if you receive false alarms and you have followed the advice in the Troubleshooting section for false alarms. Make a note of the position of the sensitivity dial before you change it should you wish to return to that setting.

To increase the sensitivity level, start by increasing it by half a level. For example, if the level is currently at 2.5, increase it to 3. Use the monitor as you normally would. If you still experience false alarms, increase it by another half level. If you are still experiencing false alarms when you have reached level 5, please contact our Help Desk at 1-888-232-6476, Mon-Fri from 9:00AM-5:00PM EST.

Because the Sensor Pad may detect movement from many sources from both inside and outside your baby's room, be sure all vibration sources are eliminated before changing the movement sensitivity level. Check to see if the Sensor Pad is resting on a flat, firm surface. Place the crib near a solid supporting wall and away from strong air currents. Do not use crib mobiles or accessories that may cause vibrations. Avoid contact with the crib when monitor is in use.

To make sure you have avoided all external sources of vibration, take your baby out of the crib while the monitor is on. If the alarm does not sound after you take your baby out of the crib, the sensitivity level may need to be decreased. To decrease the sensitivity level, adjust by half a level by going from 2.5 to 2. If you are still experiencing this problem when the level is set to 1, please contact our Help Desk at 1-888-232-6476, Mon-Fri from 9:00AM-5:00PM EST.

## Sound Lights

The Parents' Unit features Sound Lights so you can "see" your baby's sounds if background noise prevents you from hearing your baby. As the sounds get louder, more lights illuminate.

## Optional "Tic" Feature (See fig. 21)

When the "Tic" feature is selected, the Parents' Unit will continuously tick as your baby's movement is detected by the Sensor Pad. In addition, the "Tic" symbol will swing as it senses movement. If no movement is detected, the ticking will stop and the swing of the symbol will stop. If you do not want to hear the ticking, turn the "Tic" feature off.

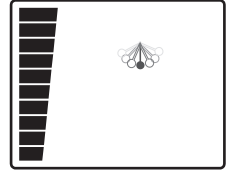
fig. 21

## Movement Indicators

There is a Movement Indicator on both the Nursery Unit and the Parents' Unit. When the Movement feature is selected, the Movement Indicator Light on the Nursery Unit will flash green as it senses your baby's movement. The flashing green light will let you know that movement is being detected. This will allow you to check on your sleeping baby in the dark without disturbing her.

On the Parents' Unit, the Movement Indicator symbol is the same as the "Tic" symbol. When the Movement feature is selected, this symbol will appear on the LCD and will swing as it senses your baby's movement (see fig. 21).

## Temporary Mute

If you want to temporarily mute the sound on the Parents' Unit, press the Power button once. The Sound symbol will flash continuously and the sound will automatically reactivate after two minutes. To reactivate the sound before the two minutes are up, press the Power button again. Be sure to only press the button down for one second, as pressing it for three seconds will turn the Parents' Unit off.

## Parents' Unit Locator

If you need to locate or page the Parents' Unit, press the Page button located on the top of the Nursery Unit. The Nursery Unit symbol will flash and the Parents' Unit will continuously beep for one minute. To stop the beep before the minute is up, press the Power button on the Parents' Unit for one second.

## Nursery Nightlight

The Nursery Unit features a soothing nightlight. Use the switch on the right to turn it on and off.

## Parents' Unit and Nursery Unit Connection

There may be times when the Parents' Unit and Nursery Unit lose connection with each other. However, if connection is lost, the Movement Sensor Pad will continue to monitor your baby and the alarm will sound in the nursery. To reestablish the connection between the units, press the Menu button on the Parents' Unit for two seconds. The Out-of-Range symbol will flash and a beep will sound. Then press the Page button on the top of the Nursery Unit for two seconds. The green Movement Indicator Light will flash once, a beep will sound, and the units will automatically reconnect. When the connection is complete, the Out-of-Range symbol will stop flashing and the Parents' Unit will beep once.

## TIPS FOR USING YOUR MONITOR


- When the monitor is in use, do not use a crib mobile or other crib accessories that may produce vibrations. Other sources of vibrations may include fans, washing machines, loud music, etc. These sources of vibration may interfere with the performance of your Angelcare® monitor and the alarm will not sound if it continues to detect any movement other than your baby's. Be sure all vibration sources are eliminated before using the monitor.
- You may experience false alarms for several reasons. The most common reason is the Nursery Unit has not been turned off after you have removed your baby from the crib. Another reason is the Sensor Pad has not been placed under the mattress properly. Be sure the Sensor Pad is on a flat, firm surface with the turquoise side facing up. The sensitivity level may also need to be adjusted. Refer to the Movement Sensitivity Adjustment section on Pg. 12.
- Your monitor can be used in any crib that is stationary and designed to safely hold a baby. If your crib has a spring or slat base you will need to place a piece of Masonite or plywood under the Sensor Pad. This support must be at least 13 inches long on all four sides and 1/4 inch thick. *Only use your Angelcare® monitor in a stationary sleeping environment such as a crib where the Sensor Pad can rest on a completely flat, firm surface.*
- When unplugging the AC adapter from the Nursery Unit, be sure it is switched to the OFF position. If it isn't, the unit will revert to battery mode if batteries are installed.
- You can use rechargeable batteries in the Nursery Unit, however, they will not be recharged by plugging in the AC adapter. To recharge, remove the batteries and use an external charger.
- **IMPORTANT:** Only use rechargeable batteries in the Parents' Units. **DO NOT** use alkaline batteries. Using alkaline batteries will damage the Parents' Units.

## UNDERSTANDING THE SYMBOLS

Symbol	Name	Meaning
	Nursery Unit	Appears when options are being set for the Nursery Unit. Appears when no movement is detected. Flashes when Nursery Unit is paging Parents' Unit.
	Nursery Unit and No Channel	Nursery Unit flashes and "--" is displayed when Parents' Unit is on but Nursery Unit is off.
	Channel	Indicates to which channel the monitor is set.
	Temperature Display	Indicates temperature of nursery.
	Parents' Unit	Appears when options are being set for the Parents' Unit.
	Sound	Appears when Sound feature is selected. Flashes when the Parents' Unit is temporarily on mute.

<b>Symbol</b>	<b>Name</b>	<b>Meaning</b>
	<b>Movement</b>	Appears when Movement feature is selected.
	<ul style="list-style-type: none"> <li>• “Tic”</li> <li>• <b>Movement Indicator</b></li> </ul>	Appears when “Tic” feature is selected. Swings when movement is detected regardless of whether or not “Tic” is on.
	<b>Out-of-Range</b>	<p>If Activated</p> <ul style="list-style-type: none"> <li>• Out-of-Range symbol appears.</li> <li>• Appears when Nursery and Parents’ Units are within range.</li> <li>• Flashes when they are out of range.</li> <li>• Flashes when Nursery Unit shuts down due to low battery.</li> <li>• Flashes when the channel is being changed.</li> <li>• Flashes when Nursery and Parents’ Units are reconnecting.</li> </ul> <p>Symbol will flash even if feature is not activated when:</p> <ul style="list-style-type: none"> <li>• The channel is being changed.</li> <li>• The Nursery and Parents’ Units are reconnecting.</li> </ul>
	<b>Voice Activation</b>	Appears when Voice Activation is on. Flashes when its sensitivity level is being set.
	<b>Sound Alarm</b>	Appears when Sound alarm is on.
	<b>Vibration Alarm</b>	Appears when Vibration alarm is on.
	<b>Low Temperature</b>	Appears when Low Temperature alert is on. Flashes when nursery temperature is below set temperature range.
	<b>High Temperature</b>	Appears when High Temperature alert is on. Flashes when the nursery temperature is above set temperature range.
	<b>Battery Level</b>	Indicates battery level of Parents’ Unit. Flashes when Parents’ Unit is being charged.
	<b>Battery and Nursery Unit</b>	Nursery Unit symbol appears and Battery symbol flashes when the battery level in the Nursery Unit is low.
	<b>Level Bar</b>	Sound Lights indicating baby’s movement. Displays volume level of speaker in Parents’ Unit when setting up. Displays Voice Activation sensitivity level when setting up.
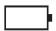
## SUMMARY OF ALARMS

Display	Alarm	What It Means
	<b>Continuous beeping and/or vibration</b>	Sensor Pad does not detect movement.
	<b>Two beeps sound every 8 seconds.</b>	Parents' Unit is out of range of the Nursery Unit or the Nursery Unit is off.
	<b>Three beeps sound every 4 seconds.</b>	Nursery temperature is below the set temperature range.
	<b>Three beeps sound every 4 seconds.</b>	Nursery temperature is above the set temperature range.
	<b>A short beep sounds every 15 seconds.</b>	Battery level is low on Parents' Unit.
	<b>A short beep sounds every 15 seconds.</b>	Battery level is low on Nursery Unit.
	<b>Continuous beeping.</b>	Nursery Unit is paging the Parents' Unit.

## IMPORTANT SAFETY INSTRUCTIONS

1. This product is **NOT** a substitute for direct supervision of your baby. Check your baby's activities at regular intervals. Monitoring of premature babies, or those considered to be at risk, should always be carried out under the supervision of a health professional.
2. Always place both the Nursery Unit and Parents' Unit on a flat surface in an upright position and allow for proper ventilation.
3. Keep adapter cords and all units out of the reach of children.
4. Avoid damage to the Sensor Pad cord and to the adapter cords. Route cords so that they are not likely to be walked on or pinched by items placed on or against them.
5. Do not place this monitor where it can get wet such as in bathtubs, sinks, swimming pools, etc.
6. Do not place this monitor near sources of heat such as radiators, stoves, or other appliances.
7. Use **ONLY** AAA rechargeable batteries and/or the AC adapters provided for the Parents' Units.
8. Use AAA rechargeable or alkaline batteries and/or the AC adapter provided for the Nursery Unit.
9. This monitor uses public airwaves to transmit signals. The Parents' Unit may pick up signals or interference from other nursery monitors in your area. Your monitor may even be picked up by other households. To protect your privacy, make sure both units are turned off when not in use. If you experience interference, change the channel on both units.

10. This monitor is perfectly safe. The Sensor Pad and its cord are entirely passive.

## CARE AND CLEANING

Disconnect all units and remove all batteries before cleaning. Clean the exterior surface of the units with a soft dry or slightly damp cloth. Do not spray with cleaning products or solvents. Do not submerge in water or any other liquid.

During long periods of non-use remove all batteries from their compartments and unplug the AC adapters.

## TROUBLESHOOTING

<b>Problem</b>	<b>Possible Cause</b>	<b>Solution</b>
<b>False alarms.</b>	Baby was removed from crib and the Nursery Unit was not switched off.	Turn the Nursery Unit off.
	The turquoise side of the Sensor Pad is not facing up.	Make sure the turquoise side is facing up.
	The Sensor Pad is not detecting your baby's movements because the sensitivity level is too low.	The sensitivity level may need to be increased. Refer to the Movement Sensitivity Adjustment section on Pg. 12.
	You are using a crib wedge.	A crib wedge may affect the operation of the Sensor Pad. Remove wedge.
	Sensor Pad is not in full contact with the crib mattress or is not resting on a flat, firm surface.	Make sure there is no thick bedding between the mattress and the Sensor Pad. The Sensor Pad must rest on a firm surface. If the crib has a spring or slat base, place a piece of plywood or Masonite measuring at least 13" on all four sides and 1/4" thick.
	Baby has moved far away from the Sensor Pad.	Make sure the Sensor Pad is positioned correctly in the crib. Sensitivity level may need to be adjusted. Refer to the Movement Sensitivity Adjustment section on Pg. 12.

<b>Problem</b>	<b>Possible Cause</b>	<b>Solution</b>
	Sensor Pad cord is not connected properly to the Nursery Unit.	Unplug and reconnect the cord.
<b>Movement alarm will not sound.</b>	Nursery Unit is detecting movement when crib is being touched.	Avoid contact with the crib when monitor is on.
	Nursery Unit is detecting movement from outside the crib such as motorized devices, strong drafts, etc.	Place crib near a solid supporting wall to avoid Sensor Pad from detecting movement from outside the crib. Turn off crib mobile, fans, loud music or anything else that may cause vibrations.
	Sensor Pad sensitivity level is set too high.	You may need to decrease the sensitivity. Refer to the Movement Sensitivity Adjustment section on Pg. 12.
	Movement function is not selected.	Make sure the Movement function is selected. The Movement symbol should appear.
	Units have lost connection with one another.	Reconnect units. Refer to the Parents' Unit and Nursery Unit Connection section on Pg. 13.
<b>Vibration Alarm is selected but is not working.</b>	The batteries are low in the Parents' Unit.	Check or charge batteries in Parents' Unit.
	The Parents' Unit is in the Charging Base.	The Vibration Alarm does not work when the Parents' Unit is in the Charging Base. Take the Parents' Unit out of the Charging Base.
<b>No sound is being transmitted or received.</b>	Sound function is not selected.	Make sure the Sound feature is selected. The Sound symbol should appear.
	Voice Activation is on and the sensitivity level is too low.	Increase the sensitivity level.

<b>Problem</b>	<b>Possible Cause</b>	<b>Solution</b>
	Parents' Unit is temporarily muted.	Press the Power button to deactivate the mute.
	Units have lost connection with one another.	Reconnect units. Refer to the Parents' Unit and Nursery Unit Connection section on Pg. 13.
	Batteries are low or improperly installed.	Check or replace batteries.
	AC adapters are not connected properly or electrical outlet is out of order.	Check connections or change to another electrical outlet.
	Either the Parents' Unit or Nursery Unit is not on.	Make sure both units are on.
	Volume level is too low.	Increase volume level.
<b>Out-of-Range symbol is flashing and Parents' Unit emits 2 beeps every 8 seconds even when the units are within range.</b> 	Nursery Unit has automatically shut down due to low battery.	Check or replace batteries, or use AC adapter.
<b>After the movement sensor alarm has sounded and been turned off on the Parents' Unit and the Nursery Unit has been powered off, the Parents' Unit continues to beep.</b>	The Out-of-Range Indicator is alerting you that the Parents' Unit is on while the Nursery Unit is off.	Either turn off the Out-of-Range Indicator feature to avoid this from happening or turn the Nursery Unit back on.
<b>Parents' Units options are not changing when they are being reset.</b>	Select button is not being pressed.	Select button must be pressed when desired option appears.
<b>Units do not turn on.</b>	Batteries are low or improperly installed.	Check or replace batteries.
	AC adapters are not connected properly or electrical outlet is out of order.	Check connections or change to another electrical outlet.
<b>There is very weak reception.</b>	Parents' and Nursery Units are too far apart.	Bring Parents' Unit closer to Nursery Unit. Activating the Out-of-Range feature will tell you the units are too far apart.

<b>Problem</b>	<b>Possible Cause</b>	<b>Solution</b>
	One or both units are placed near a large metal mass.	Change the position of one or both units.
	One or both units are not in the upright position.	Place units on flat surface away from baby's reach.
	Batteries are low.	Check or replace batteries.
<b>There is static, distortion, interference, etc.</b>	Parents' and Nursery Units are too far apart.	Bring Parents' Unit closer to Nursery Unit. Activating the Out-of-Range feature will tell you the units are too far apart.
	There may be interference from other electrical equipment.	Change the channel on Parents' Unit.
	Parents' Unit is located near motorized devices, fluorescent lights, television, etc.	Rotate Parents' Unit or move it away from source of interference.
<b>There is feedback (squealing noises).</b>	Units are too close to each other.	Move units farther apart (at least 10 feet apart).
	Volume is too high.	Decrease volume on Parents' Unit.

## **TECHNICAL SPECIFICATIONS**

No. of Channels: 8

Power Supply: Four AAA alkaline or rechargeable batteries, or 7.5V AC adapter for Nursery Unit.

Four AAA rechargeable batteries for each Parents' Unit and 7.5V AC adapter for Charging Base for each Parents' Unit.

Frequency: 900 MHz for sound. 2.4 GHz for data.

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) This device must accept any interference received, including interference that may cause undesired operation.

This equipment has been tested and found to comply with the limits for Class B Digital Device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

Any changes or modifications not expressly approved by the party responsible for compliance may void the user's authority to operate the equipment.

## LIMITED PRODUCT WARRANTY

**Register your product warranty online at [www.bebesounds.com](http://www.bebesounds.com).**

Unisar Inc., the distributor of the Bébésounds® Angelcare® Deluxe Movement Sensor with Sound Monitor, warrants to the original owner of this product that this product is free from defects in materials and workmanship for a period of one (1) year from the date of the original purchase. If this product fails to function properly when used as directed under normal conditions within the 1-year period, Unisar Inc. will either repair or replace the product, at its option, free of charge, when it is returned in its original packaging, postage prepaid, with proof of purchase. Unisar Inc. will bear the cost of repairing or replacing the product and shipping it back to you within the United States. Repair or replacement of the defective item is subject to verification of the malfunction or defect when delivered to:

Unisar® Service Center  
15 West 36<sup>th</sup> Street  
New York, NY 10018

**NOTE:** The warrantor expressly disclaims liability for incidental, consequential, or special damages of any nature. This warranty does not cover 1) A defect that has been caused by repair or modification by any person other than a Unisar® technician; 2) A defect caused by negligence, accident, or misuse of the product; 3) A defect caused by incorrect installation of and/or defective batteries; 4) Battery life; 5) A defect caused by the use of any adapter not provided by the warrantor; or 6) Damage resulting from improper operation.

LEGAL RIGHTS VARY FROM STATE TO STATE.

**NOTE:** This warranty is valid only for products sold in the United States. If you have purchased this product outside of the United States, please see your local retailer or distributor for service.

If you have any questions, please call the Bébésounds® Help Desk toll free at 1-888-232-6476 Mon-Fri from 9:00AM-5:00PM EST.

### Angelcare Masonite Board

If your crib does not have a flat bottom for the Sensor Pad to rest on, place the Masonite on top of the springs or slats and the Sensor Pad on top of the board and under the mattress. This will allow the Sensor Pad to operate properly.

### NasalClear® Aspirator

Clear stuffy noses FAST!™ Safely and quickly clear your baby's nasal congestion with this battery operated aspirator. Gentle enough to use on a newborn baby. Comes apart for easy and thorough cleaning.

### LiteUps™

Nail Clippers, Tweezers, and Scissors feature a bright LED light so you can see your baby's nails and skin more easily. Cut your baby's nails in the dark while she sleeps. Tweezers also magnify skin so it is easier to remove splinters.

### One-Second Ear Thermometer

Makes taking your child's temperature quick and convenient. No probe cover or "ear-tug" required. Cleans with alcohol. Lighted jumbo display. Measures in Fahrenheit and Celsius.

Call 1-800-430-0222 or  
visit [www.bebesounds.com](http://www.bebesounds.com)  
to find the store nearest you.

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**Unisar Inc.** 15 West 36th Street, New York, NY 10018 • 1-800-233-1196 • [info@unisar.com](mailto:info@unisar.com) ©2007 Listro Associates