# BébéSounds®

## Fever Monitor Manual

Model #BR143

User's Manual

## INTRODUCTION

Thank you for purchasing the BébéSounds® Fever Monitor. This Fever Monitor will enable you to continuously monitor your baby's fever without having to disturb her. Use the Fever Monitor when your baby is not feeling well and is feverish, or if your child is prone to extreme spikes in temperature.

The Fever Monitor keeps track of your baby's temperature from the surface of the skin. It clips to the diaper or pajamas with the sensor probe resting against the skin. Be sure the sensor probe is touching your baby's skin and that your baby is wearing clothing that covers the stomach as well as the monitor to avoid the loss of body heat.

The Fever Monitor will alert you if your baby's fever spikes up one degree. For example, if your baby's temperature is 100°F (37.8°C) and goes up to 101°F (38.3°C), the Fever Monitor will continuously beep until you reset it. If the Fever Monitor loses contact with your baby's skin it will also beep continuously until you readjust it. These beeps are gentle enough not to wake your baby but loud enough to be picked up by your sound monitor so you can sleep more easily when she is not feeling well.

You may also use the probe of the Fever Monitor as an oral thermometer to get an accurate temperature reading and to compare the temperature registered from the skin with the temperature registered with a more conventional method. Remember to clean the probe with alcohol with each use.

Use of the Fever Monitor should not replace a visit to your doctor. It is recommended that you advise your doctor any time a temperature is above 100°F (37.8°C).

**NOTE:** This product is intended for use with children under 8 years old.

**WARNING:** This product is not a toy. Do not allow children to play with it. Keep out of children's reach when not in use. Always use under adult supervision.

If you have any questions about our Fever Monitor, please call our tech support line toll free at 888.232.6476 Monday–Friday, 9:00AM-5:00PM EST.


## KEY FEATURES

- Your baby's fever is continuously monitored, even while you sleep
- You can check her fever level without disturbing her
- The monitor beeps if temperature rises 1°F
- The monitor beeps if the probe loses contact with the skin
- Beeps can be picked up by your sound monitor but are soft enough so they don't wake your baby
- Can also be used as an oral or underarm thermometer
- Water-resistant
- Non-invasive
- Battery life: 6 Months
- Easy-to-read LCD display
- Low battery warning
- Measures in Fahrenheit


## INSTRUCTIONS FOR USE AS A FEVER MONITOR

1. Clean the probe with alcohol.
2. Press the Power button and listen for one beep and release. This indicates the Fever Monitor is ready to start taking a temperature reading.
3. After approximately 10 seconds it will beep twice and the horn icon will flash. The icon will stop flashing after approximately 15 seconds indicating it is ready to enter

## Remove protective film before first use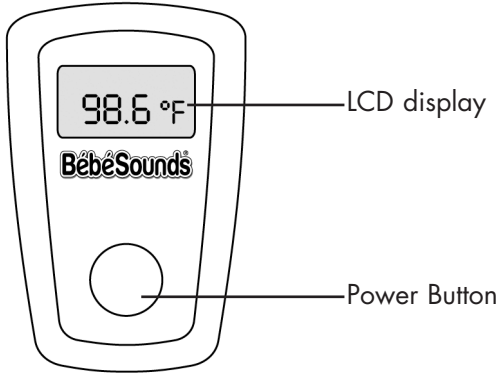

Front View

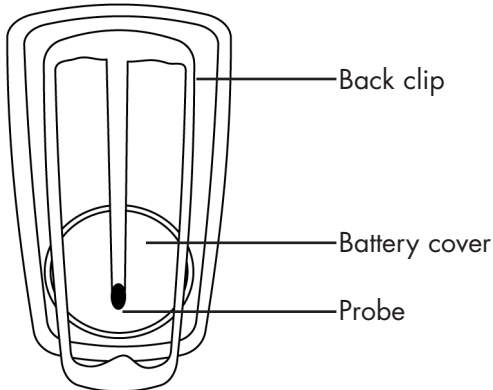Back View

- the monitoring mode.
4. Attach the Fever Monitor to your child's diaper or pajamas making sure the sensor probe is against her bare skin and that your baby is wearing clothing that covers the stomach as well as the monitor to avoid the loss of body heat. It will take approximately 3 minutes for the reading to stabilize.
  5. To reset the monitor after the alarm goes off, press the Power button once and the beeping will stop. This will not interrupt the monitoring feature. Therefore, if your baby's fever goes up another degree, the monitor will beep again.
  6. To turn off, press the Power button for 2 seconds.

**NOTE:** To prolong battery life, please turn off your Fever Monitor when not in use. Store in a cool dry place.

## INSTRUCTIONS FOR USE AS AN ORAL THERMOMETER

1. Pull the monitor probe up and away from the monitor and wipe it clean with alcohol and then with water.
2. Press the Power button and listen for 1 beep which indicates it is ready to start taking a temperature reading.
3. Place the monitor probe under your child's tongue toward the back. The temperature reading will be complete in approximately 1 minute.

## FEVER INFORMATION

What is a "normal" temperature?

Temperature varies with each individual. A normal temperature refers to each individual's average temperature while they are well. Normal average temperatures range from 96.6–99.5° F (35.9–37.5°C) when taken orally. Oral temperature readings are more accurate than skin surface

readings. Temperatures taken from the skin's surface may be 2-3°F (1-1.5°C) lower than oral readings. For example, if the Fever Monitor registers a skin temperature of 97°F (36.1°C), your child's oral temperature may actually be 99-100°F (37.2-37.8°C). This differential should be considered when monitoring your baby's temperature. We suggest you take your baby's temperature using either the Bébésounds® Ear Thermometer or the Bébésounds® 5-Second 3-Way Thermometer to determine the difference between the reading from the skin and the reading using another method.

#### Normal Temperature Ranges

Oral	96.6-99.5°F	(35.9-37.5°C)
Ear	96.6-99°F	(35.9-37.2°C)
Rectal	97.7-100.3°F	(36.5-37.9°C)

**CAUTION:** Do not remove the clip on the back of the monitor. Only use the required battery type (1.5V A76 alkaline battery). Do not immerse in water. Only use alcohol or water to clean the probe. Keep out of the reach of children when not in use. Do not expose to direct sunlight.

#### CARE AND CLEANING

1. The Fever Monitor is a delicate device. To ensure proper functioning, do not immerse the unit in water and avoid dropping it.
2. Clean the unit with a soft cloth moistened with either alcohol or water. Never immerse it in water or use an abrasive cleaner.
3. Keep out of the reach of children when not in use. The optimal storage temperature is between -4-140°F (-20-60°C).
4. Do not disassemble the monitor or remove the clip on

the back.

#### APPROVALS AND APPLIED STANDARDS

ASTM E112: Standard Specification for Electronic Thermometer for Intermittent Determination of Patient Temperature  
EN 606 1-1: Medical electrical equipment – Part 1 General Requirement for Safety

#### TECHNICAL DATA

- For household use only
- Temperature readings are to the nearest 0.1°F
- Temperature measurement range: 50-113°F
- Ambient operating environment: 50-100.4°F
- Temperature accuracy:  
50-90°F ± 6°F  
90-113°F ± 0.2°F
- Each unit requires a 1.5-volt A76 alkaline battery (included)
- Battery life: 6 Months

#### LIMITED PRODUCT WARRANTY

Unisar Inc., the distributor of the Bébésounds® Fever Monitor warrants to the original owner of this product that this product is free from defects in materials and workmanship for a period of one (1) year from the date of the original purchase. If this thermometer fails to function properly when used as directed under normal conditions within the 1-year period, Unisar Inc. will either repair or replace the product at its option, free of charge, when it is returned in its original packaging, postage prepaid, with proof of purchase. Unisar® will bear the cost of repairing or replacing the product and shipping it back to you. Repair or replacement of the defective item is subject to verification of the malfunction or defect when delivered to:

**Unisar® Service Center**  
**15 West 36th Street**  
**New York, NY 10018**

**NOTE:** The warrantor expressly disclaims liability for incidental, consequential, or special damages of any nature. This warranty does not cover 1) A defect that has been caused by repair or modification by any person other than a Unisar® technician; 2) A defect caused by negligence, accident, or misuse of the product; 3) A defect caused by incorrect installation of and/or defective batteries; 4) Battery life; or 5) Damage resulting from improper operation.

#### **LEGAL RIGHTS VARY FROM STATE TO STATE.**

The Fever Monitor is a personal care product and cannot be returned to the retailer. All exchange and repair requests should be directed to the BébéSounds® Help Desk toll-free at: 1-888-232-6476

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©2005 Listro Associates